

SWASH[®] GOLD GLOVES – FRAGRANCE FREE

GENERAL INFORMATION

Article number	A04070-5
Basic UDI-DI	PP10417BODY.WASH95
UDI-DI	111666482521
Intended use	Intended for a complete body wash of incontinent and/or bedridden patients as an infection prevention measure.
Content of package	5 gloves
Content of box	60 packages
Manufacturer	Arion Manufacturing B.V. – The Netherlands Rutherford 11, 6422 RE Heerlen
Issue date	07/2020
Revision date	N.A.

PRODUCT INFORMATION

Type of product	Medical device class 1 (2017/745/EC)
Composition cloth	Polyester & Lyocell
Production method cloth	Spunlace
Thickness cloth	ca. 82 GSM
Box weight	ca. 6075 gram
Total net waste weight cloths	ca. 37.5 gram
Total net waste weight foil	ca. 5 gram
pH-value	5 – 6
Dimensions cloth	14 cm x 22 cm
Dimensions package	11 cm x 21,5 cm

INGREDIENTS LOTION

PURPOSE

CONCENTRATION %

Aqua	Solvent in which the other ingredients are dissolved.	75-100
Propylene Glycol	Water-attracting ingredient that reduces dehydration of the skin. Next to that, it is a solvent and it determines the viscosity of the lotion.	1-5
Glycerin	Water-attracting ingredient that reduces dehydration of the skin. Also used as a solvent in many cosmetic products like shampoos.	1-5
Caprylyl Glycol	Skin conditioner. Also has preservative properties.	0.1-1
Polysorbate 20	Emulsifier. Binds the lotion and stabilizes the emulsion.	0.1-1
Disodium Cocoamphodiacetate	Mild cleaning ingredient with skin treating properties.	0.1-1
Tocopherol (Vitamin E)	Preservative and moisturizer to prevent dehydration of the skin.	0.1-1
Glyceryl Caprylate	Humectant. Also has preservative properties.	0.1-1
Potassium Sorbate	Natural preservative.	0.1-1
Dipropylene Glycol	Solvent for the preservative system.	0.1-1

Sodium Chloride	Additive.	<0.1
-----------------	-----------	------

INSTRUCTION FOR USE	Use 1 glove for each part of the body, as indicated below:
	<ol style="list-style-type: none"> 1. Face, neck and chest 2. Right arm, left arm and armpits 3. Abdomen and genital area 4. Right leg, left leg and feet 5. Back and buttocks
	If necessary, heat the Swash [®] package before use.
	Check the temperature of the gloves of a heated package before use.
	The gloves are disposable and each package should be used for one person only.
	Skin creases must be patted dry using a towel to prevent moisture-related skin lesions.
	Only suitable for external use.
Nursing staff is advised to use up a packet of Swash [®] within 48 hours after opening.	
Do not flush down the toilet.	

TEST RESULTS	
Eye irritation test	Results show that the lotion does not cause any irritation of the cornea and the iris. In addition, the conjunctiva do not swell or turn red.
Skin irritation test	Results show that the Swash [®] Gold Gloves lotion does not cause any irritation of the skin. There was no sign of edema or red skin during or after the test.